

Omega Laboratories Now Offers Q.E.D.



The “On the Spot Leader” in Alcohol Testing

Why Should You Choose Q.E.D.?

Fast and Simple Testing Process

Provides quantitative results in less than five minutes using only a saliva sample.

Accurate & Quantitative Results

Offers results similar to blood-based test and a thermometer-like reading, leaving no room for misinterpretation of results.

Built-in QA Spot™

Assures quality control for every test by ensuring test administrators apply specimen in the correct location with the correct quantity of specimen.

Low Cost

Provides a lower cost per test option than breath or blood alcohol.

Department of Transportation Approved

Delivers CLIA-waived and DOT-approved alcohol testing that can be administered by any trained professional.

Q.E.D Collection Procedure



Step 1: Swab mouth to collect saliva

Actively swab around the cheeks, gums, and under the tongue for 30-60 seconds until the cotton swab is thoroughly saturated.



Step 2: Insert collector into test

Place the Q.E.D.® test on a flat surface. Gently twist the collector into the entry port.



Step 3: Read color bar

Allow two (2) minutes for the Q.E.D.® test to develop. The QA Spot™ must be dark purple to indicate a valid test.



Step 4: Verify Result

Verify any presumptive positive results in accordance with your organizations testing policy (typically utilizing a breath alcohol testing service).